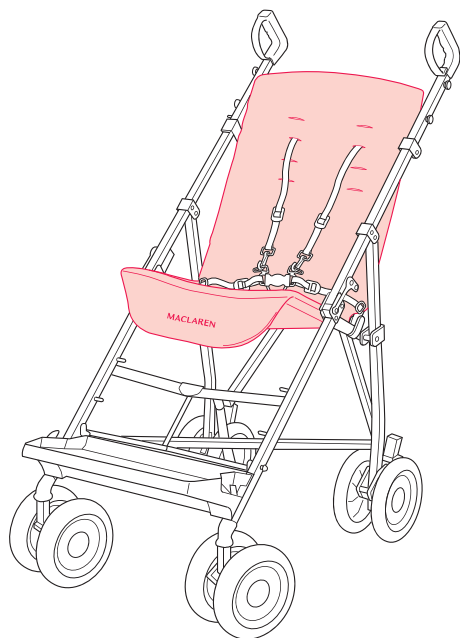
MACLAREN

major elite

EN **Padded Seat Cover**
FR **Hamac rembourré**
ES **Cubierta acolchada
del asiento**
DE **Wattierter Sitzbezug**

IT **Copri sedile imbottito**
NL **Gewatteerde
zittinghoes**
CZ **Polstrovaný potah
sedačky**

EN TO FIT THE PADDED SEAT COVER

Release harness clasp and unclip shoulder straps.

- 1A** Thread shoulder harness buckles through holes in seat back.
- 1B** Thread crotch strap buckle up through hole in seat base.
- 2C** Thread shoulder harness buckles through holes in padded seat cover and corresponding holes in seat back. The buckles must pass through both padded seat cover and transport chair seat back.
- 2D** Thread crotch strap buckle down through hole in padded seat cover and corresponding hole in seat base. The buckle must pass through both padded seat cover and transport chair seat base.
- 3** Fit padded seat cover pocket over seat base and attach the two poppers.
- 4** Fit padded seat cover pocket over seat back and attach the two poppers. Reattach shoulder strap clips and fasten harness clasp.

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

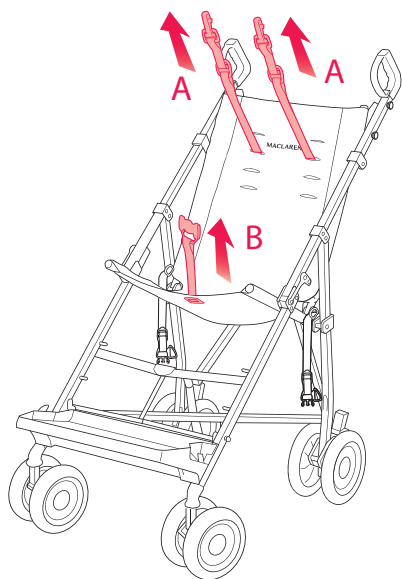
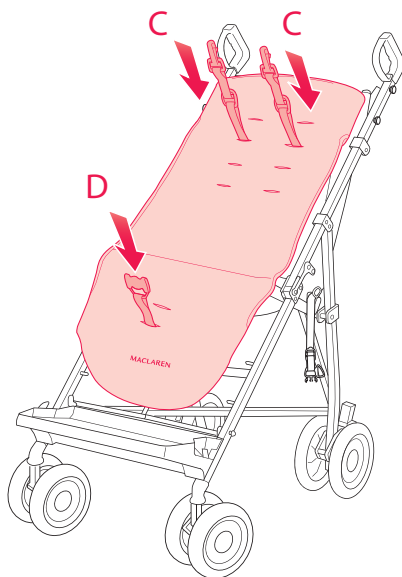
3

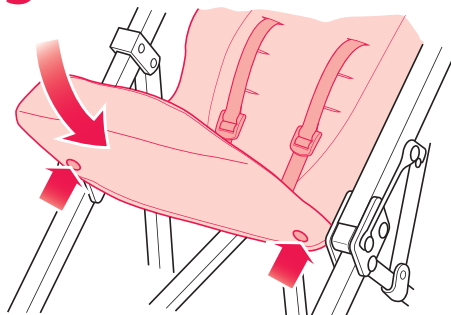4

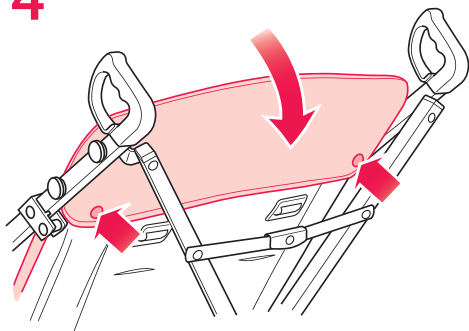EN PLEASE READ THIS INSTRUCTION LEAFLET FULLY PRIOR TO USE AND KEEP FOR FUTURE REFERENCE

- The transport chair can be folded with the padded seat cover attached.
- **CLEANING:** Sponge lightly using a damp cloth and a mild detergent OR machine wash on synthetic cycle. Dry thoroughly before reuse.
- Ensure that shoulder harness straps and crotch strap are located in the correct holes and are securely retained by the seat fabric hole reinforcements.
- Ensure that all buckles are correctly adjusted.
- For full use and safety instructions, please refer to the Major Elite Attendant User Guide.

OTHER MAJOR ELITE ACCESSORIES AVAILABLE:

Sunshade	Lateral Inserts	Chest Pad
Boot	Raincover	Shopping Basket

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