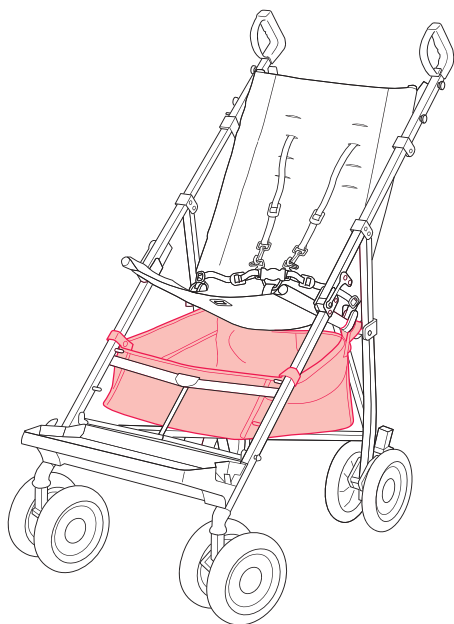
MACLAREN

major elite

EN Shopping Basket
FR Panier
ES Cesta de compras
DE Einkaufstasche

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EN TO FIT THE SHOPPING BASKET

- 1 The shopping basket is fitted below the seat base, resting on the bottom 'X' frame and secured to the front legs and seat support tubes by popper straps.
- 2 Unfold the shopping basket and place it on the 'X' frame under the seat with the lowest edge of the basket at the front. Wrap the front straps around each front leg just above the highest footrest pin and secure the poppers.
- 3 Wrap the rear straps around each seat support tube just forward of the rear leg brackets and secure the poppers.

WARNING: Maximum load 2kg (4.4lbs). Overloading may make the transport chair unstable.

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
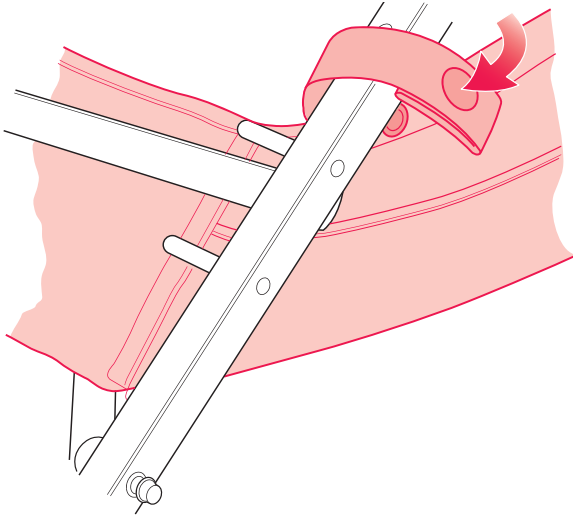
2

3

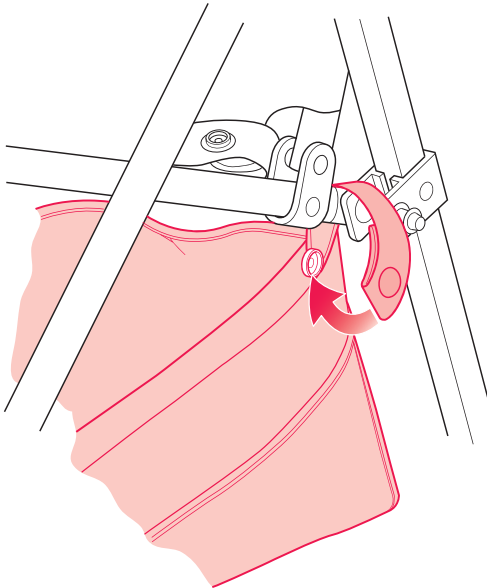EN PLEASE READ THIS INSTRUCTION LEAFLET FULLY PRIOR TO USE AND KEEP FOR FUTURE REFERENCE

- The transport chair may be folded with the shopping basket still attached but ALL CONTENTS MUST BE REMOVED BEFORE FOLDING.
- CLEANING: Sponge lightly using a damp cloth and a mild detergent OR machine wash on synthetic cycle. Dry thoroughly before reuse.
- CAUTION: Overloading may make the transport chair unstable. Maximum load: 2kg (4.4lb).
- For full use and safety instructions, please refer to the Major Elite Attendant User Guide.

OTHER MAJOR ELITE ACCESSORIES AVAILABLE:

Padded Seat Cover	Boot	Raincover
Sunshade	Lateral Inserts	Chest Pad

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