

MACLAREN


major elite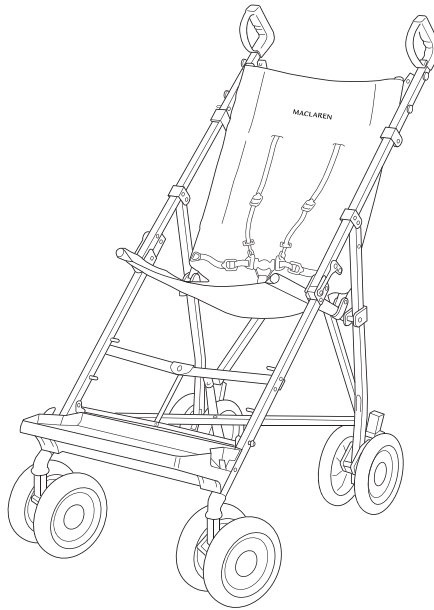


1




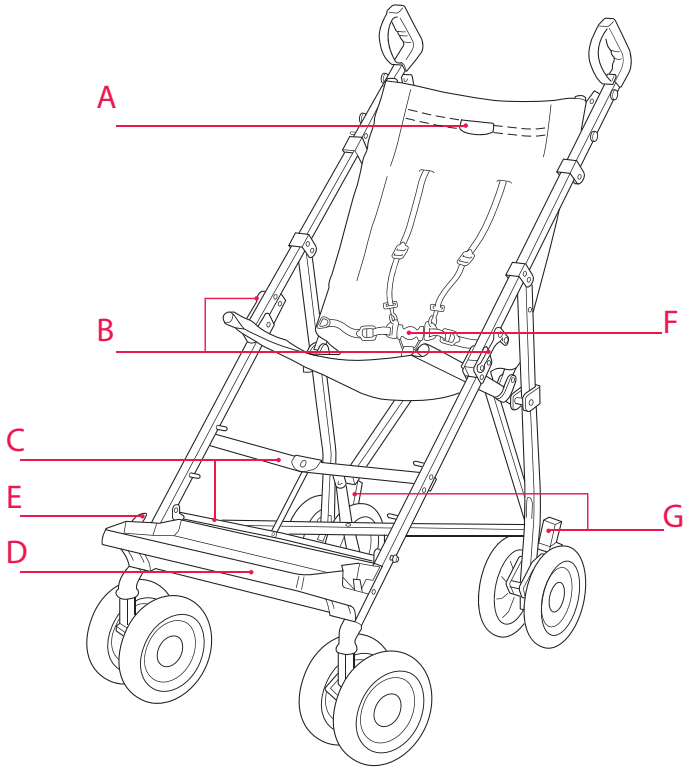


2


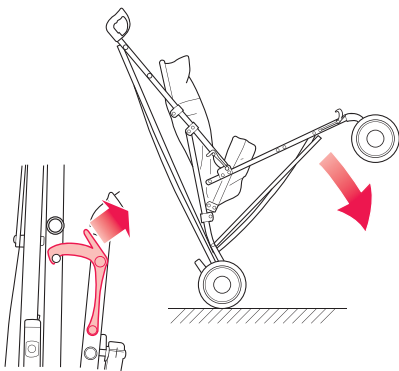
3

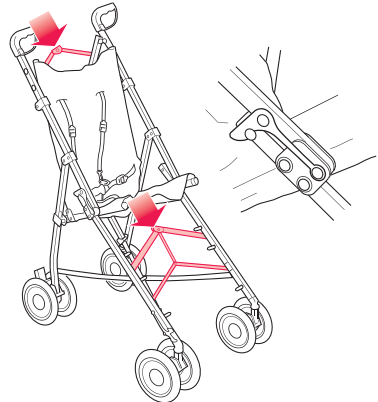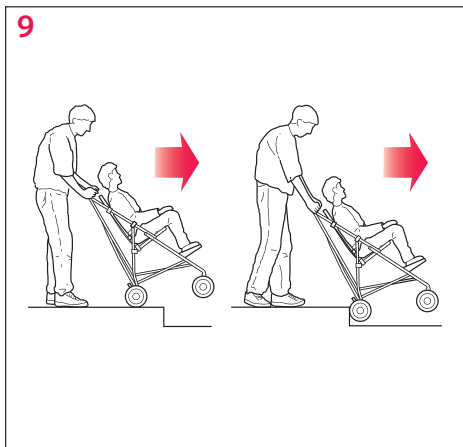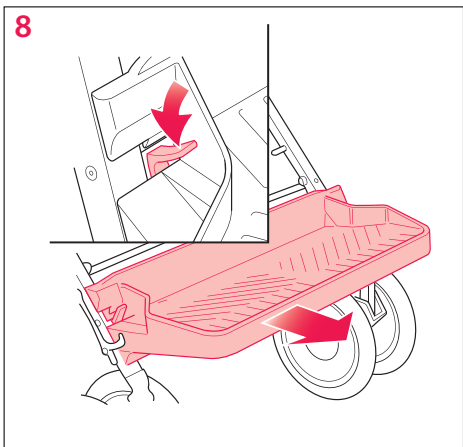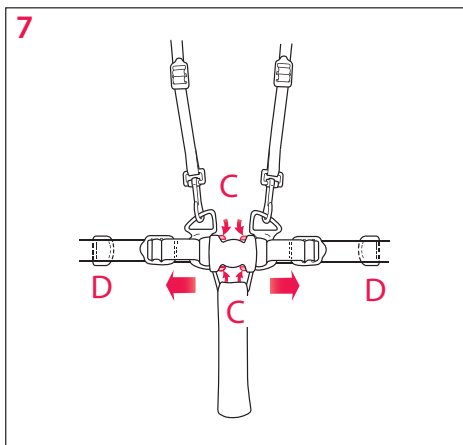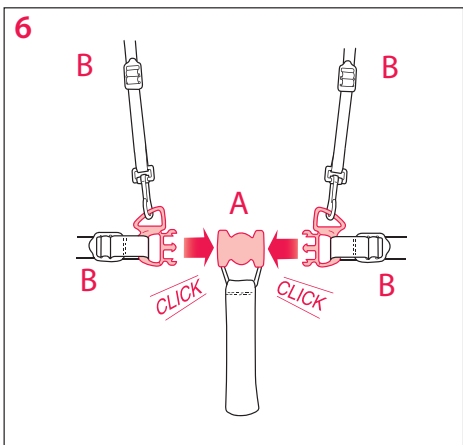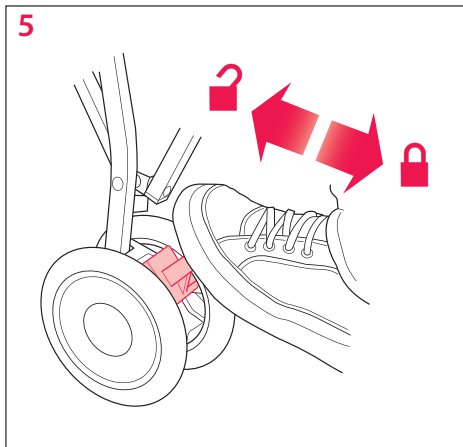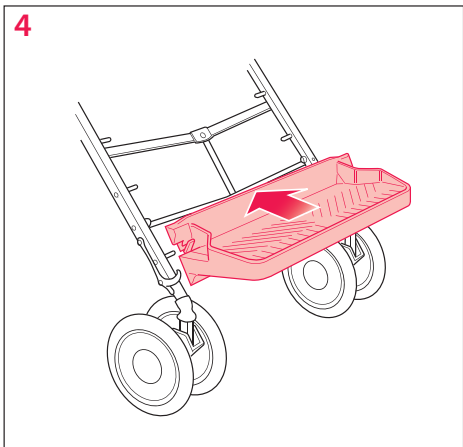
10

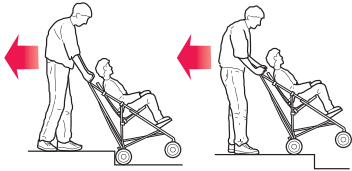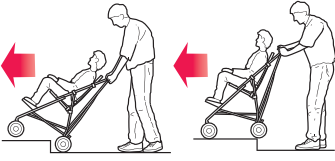
11


12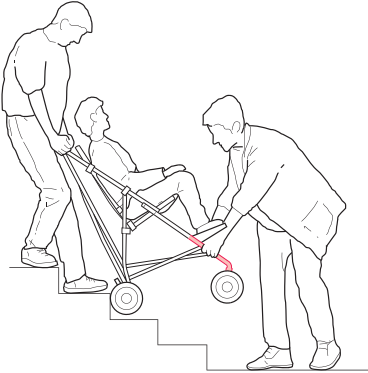


13

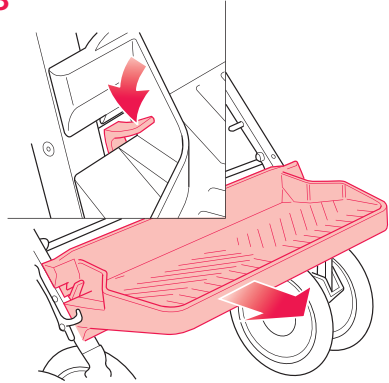14

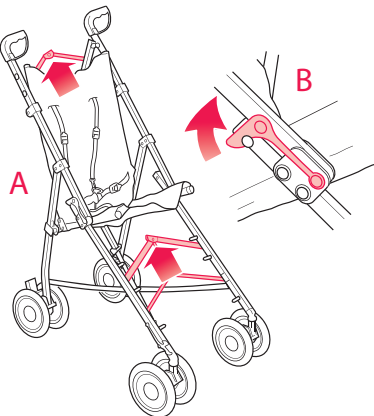15

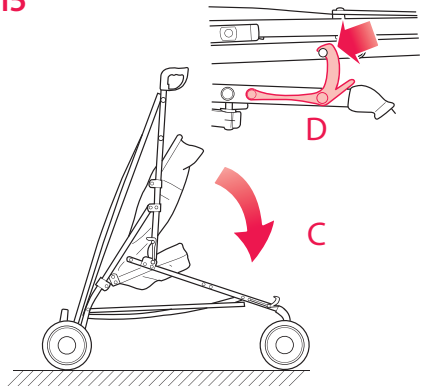Contents

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INTRODUCTION

- The Major Elite is a lightweight folding transport chair developed for journeys of short duration for children up to 50 kg/110 lbs who are without significant postural instability and need minimal support.
- Expert medical advice regarding the suitability of this product must be sought before use.
- To adapt the Major Elite transport chair for the smaller child, a pair of Lateral Inserts is available. Seek advice from the Supplier or Wheelchair Centre.
- A responsible adult attendant of adequate strength must propel and supervise the transport chair during use.
- The lightweight folding construction enables easy storage. When folded, the transport chair can easily be lifted into a car boot, either diagonally or across the width.

IMPORTANT PARTS OF YOUR TRANSPORT CHAIR

- 1A** Upper brace
- 1B** Locking clips
- 1C** Lower braces
- 1D** Footrest
- 1E** Storage clip
- 1F** Harness
- 1G** Parking device

SAFETY NOTES

Please read this instruction booklet fully prior to use and keep for future reference.

- Expert medical advice regarding the suitability of this product must be sought before use.
- The attendant should be familiar with the operation of the transport chair – if in doubt, stop and read this instruction booklet. Your Local Wheelchair Centre will be able to provide advice regarding the correct and safe use of this transport chair.
- Take note of how the child wishes to move or be moved. Co-operation between attendant and child will reduce the risk of injury occurring during lifting and manoeuvring operations.
- Do not allow the child to climb unassisted into the transport chair.
- Do not overload this transport chair with additional children or goods. Bags and other items must not be hung from the handles.
- Before setting off, ensure the child is comfortable and that clothes and rugs etc. are tucked in, so they do not catch in the wheels.
- Ensure the waist strap and shoulder harness are correctly adjusted and fitted. The attendant must know how to fit, adjust and release the buckles and clips.

- Do not stand on the footrest.
- Hold on to your transport chair when close to moving road vehicles or trains. Even with the parking device on, the draught from the vehicle may move the transport chair.
- Always apply the parking device when the transport chair is stationary.
- Exercise caution when mounting or dismounting kerbs/curbs.
- Remove the child and fold the transport chair when ascending or descending escalators.
- Never tip the transport chair forward at a kerb.
- Do not attempt to take the transport chair up or down a flight of steps, unless you and your helper are sure you have control of the transport chair, and can take the weight of its occupant.
- The Major Elite must not be used as a seat for a child whilst travelling in a motor vehicle or bus.
- Never leave the child unattended in the transport chair, even for the shortest time.
- The attendant should take precautions against fire risks. In buildings, check the location of fire exits and ensure procedures are understood.
- Do not let children play with or hang on the transport chair.
- Do not use your transport chair in a manner for which it is not designed.
- Ensure children are clear of moving parts before making adjustments.
- Care must be taken to avoid trapping fingers when opening and folding the transport chair.
- Maclaren cannot accept responsibility for the use of accessories other than those supplied by Maclaren. Any damage caused by the use of accessories not supplied by Maclaren will not be covered by the terms of our warranty.

PREPARATION FOR USE

OPENING THE TRANSPORT CHAIR

- 2** Release the storage clip and open out the transport chair.
Care must be taken to avoid trapping fingers when opening the transport chair.
- 3** Push both upper and lower braces fully down and ensure the locking clips are engaged.

FITTING THE FOOTREST

- 4** Choose the height position most comfortable for the child. Clip the footrest to the support pins in either the low, mid or high position and apply upward pressure to the footrest to ensure both clips are fully engaged.

IN USE

OPERATING THE PARKING DEVICE

- 5** TO ENGAGE THE PARKING DEVICE
Using a foot, pull back on BOTH pedals ensuring the pins are securely located in the wheel spokes.
- TO RELEASE THE PARKING DEVICE
Push forward on BOTH pedals. The parking device should not be used to slow down a moving transport chair.

HARNESS OPERATION

TO ATTACH THE HARNESS

- 6A** Insert both fasteners securely into the clasp.
- 6B** Adjust strap lengths by sliding the buckles. The belt and shoulder straps should be snug.
- TO RELEASE THE HARNESS
- 7C** Squeeze fastener prongs and pull the belt apart. It may be necessary to unclip the shoulder straps from the clasp.
- 7D** "D" rings are included for fitting an additional harness if desired. The harness may not be adequate in all cases. Expert medical advice should be sought.

REMOVING THE FOOTREST

- 8** Depress both white plastic catches located on each side of the footrest and withdraw the footrest.

GETTING INTO THE TRANSPORT CHAIR

Ensure both parking devices are ON.

To make access easier, the footrest can be removed.

If possible, place the back of the transport chair squarely against a wall or immovable object. Alternatively, another helper may hold the transport chair steady.

If the child is not being lifted into the transport chair, align the front wheels with the back wheels. Place the child's feet on the ground between the front wheels, one foot just forward of the other and lower the child gently into the seat. The child may assist by holding the transport chair's front legs.

Fasten the harness and adjust the strap lengths. Belt and shoulder harness should be snug. Replace the footrest at the appropriate height.

Take note of how the child wishes to move or be moved. Co-operation between attendant and child will reduce the risk of injury occurring during lifting and manoeuvring operations.

GETTING OUT OF THE TRANSPORT CHAIR

Ensure both parking devices are ON and the footrest is removed.

If possible, place the back of the transport chair squarely against a wall or immovable object.

Alternatively, another helper may hold the transport chair steady.

Align the front swivel wheels with the rear wheels to allow room for the child's feet.

Unfasten the harness clasp and if necessary unclip the shoulder straps.

If the child is not being lifted out of the transport chair, the child should grasp each transport chair front leg and bending slightly forward, place both feet firmly on the ground, one forward of the other. The child should then push upward to a standing position with assistance if necessary.

PUSHING TECHNIQUES

Plan a trip to take into account obstacles, terrain and distances to be covered.

The attendant should walk at a sensible speed and warn the child before carrying out manoeuvres.

The attendant should maintain firm control of the transport chair at all times using both handles.

Avoid uneven or soft ground. If this kind of terrain is unavoidable then it is sometimes easier to pull the transport chair backwards.

A child may feel safer if pulled backwards when going down a steep slope.

Negotiate swing doors in reverse, use your back to open the doors then pull the transport chair through.

Operating the transport chair should be an enjoyable experience for both attendant and child provided that there is mutual confidence and understanding.

NEGOTIATING A KERB

9 GETTING DOWN A KERB OR SINGLE STEP

Take the front wheels to the edge of the kerb.

Hold the handles firmly and tip the transport chair backwards and balance on the rear wheels.

Take the rear wheels to the edge of the kerb.

Lower the transport chair down the kerb on its rear wheels.

Never tip the transport chair forward at a kerb.

GETTING UP A KERB OR SINGLE STEP

10 METHOD 1

Take the front wheels to the kerb. With a firm grip on the handles, tip the transport chair backwards and place the front wheels on top of the kerb. Position the rear wheels at the kerb bottom and lift to the upper level.

METHOD 2

Turn the transport chair around and position the back wheels against the kerb.

Maintaining a firm grip on the handles, tilt the transport chair backward and pull the rear wheels to the upper level.

IN USE CONT'D

NEGOTIATING STEPS

Make use of ramps and lifts when they are available.

Where possible steps should be avoided. If there is no alternative to negotiating steps then ensure at least one helper is available to give assistance.

Do not attempt to take the transport chair up or down a flight of steps unless you and your helper are sure you have control of the transport chair and can take the weight of its occupant.

During this operation, the attendant and helper must be well balanced and have secure grip on the transport chair. The attendants hands must be inside the handle grips at all times.

Take rests at regular intervals.

GOING UP STEPS

- 11** Position the transport chair squarely with the rear wheels against the bottom step.

Tilt the transport chair backward and raise to the next step.

The helper stands in front, steadies and lifts the transport chair by gripping the front frame (**not the footrest**).

Continue step by step to the top.

GOING DOWN STEPS

- 12** Position the transport chair squarely with the front wheels on the top step.

Tilt the handles backward and push the transport chair forward until the rear wheels are at the top step.

The helper stands in front, steadies and lowers the transport chair by gripping the front frame (**not the footrest**).

Take the weight and lower the rear wheels down the step.

Continue step by step to the bottom.

AFTER USE

REMOVING THE FOOTREST

- 13** Depress both white plastic clips located on each side of the footrest. Withdraw the footrest from the location pins.

FOLDING THE TRANSPORT CHAIR

- 14A** Pull both upper and lower braces up.

- 14B** Unclip BOTH locking clips.

Care must be taken to avoid trapping fingers when folding the transport chair.

- 15C** Grip the handles and push forward.

- 15D** Ensure the storage clip is engaged.

CARE OF YOUR TRANSPORT CHAIR

- Your transport chair has been designed to a high standard and with correct use and maintenance will give many years of trouble free service.
- Regularly inspect the locking devices, parking devices, harness assembly, catches, footrest, joints and fixings to ensure they are secure and in full working order.
- Parking devices, wheels and tyres are subject to wear and should be replaced if necessary by a repairer approved by the transport chair supplier.
- We recommend that the transport chair is checked by an approved repairer every 12 months.
- If the transport chair is damaged or in need of repair, the approved repairer will be able to advise on what is required. Alternatively, McLaren Customer Care is available for product information.
- Only McLaren replacement parts and accessories should be used on this transport chair.
- If wheels squeak, lubricate sparingly with silicone coating. Do NOT use oil or grease based products, as this will attract dirt, which will clog the movement.
- Make sure that you are familiar with the operation of the transport chair. It should always open and fold easily. If it does not, do not force the mechanism - stop and read the instructions.
- Plastic and painted parts may be sponged clean with warm water and a mild detergent.
- The seat fabric can be sponged or scrubbed clean using a mild detergent and/or mild disinfectant.
- Never clean with abrasive, ammonia based, bleach based or spirit type cleaners.
- After exposure to damp conditions, dry off with a soft cloth and leave fully open in a warm environment.
- Do not fold up the vehicle or accessories whilst wet, or store in a damp place. Either of these will encourage mildew to form. Always dry all items before storing.
- Store your transport chair in a dry, safe place.

ACCESSORIES AVAILABLE

Padded Seat

Sunshade

Boot

Lateral Inserts

Raincover

Chest Pad

Shopping Basket

Maclaren reserves the right to make design changes to any of their products as part of their continuous development programme.

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WARRANTY INFORMATION

Maclaren Limited Warranty

The Major Elite is automatically covered by the Maclaren Limited Warranty. For the full and most up to date terms and conditions of this Limited Warranty please ensure that you:

- Visit **maclarenbaby.com**.
- Refer to the Frequently Asked Questions section of our website at **maclarenbaby.com**.
- Contact Maclaren Customer Services on the number provided
T +44 (0) 1327 841 300.

Maclaren Sovereign Lifetime Limited Warranty™

The Major Elite can be registered for the Maclaren Sovereign Lifetime Limited Warranty™, free of charge. You must register your Major Elite at **maclarenbaby.com** within 45 days of your purchase.

For the full and most up to date terms and conditions of this Warranty please ensure that you:

- Visit **maclarenbaby.com**.
- Refer to the Frequently Asked Questions section of our website at **maclarenbaby.com**.
- Contact Maclaren Customer Services on the number provided
T +44 (0) 1327 841 300.

The Maclaren Sovereign Lifetime Limited Warranty™ may not be honoured if the prescribed terms and conditions have not been satisfied.

IMPORTANT INFORMATION

ALL MACLAREN WARRANTIES DO NOT AFFECT YOUR STATUTORY (LEGAL) RIGHTS. THE BENEFITS OF ALL MACLAREN WARRANTIES ARE OFFERED IN ADDITION TO ANY CONSUMER PROTECTION LAWS OR REGULATIONS IN THE COUNTRY OR STATE OF PURCHASE (ONLY). ANY RIGHTS AND REMEDIES CONFERRED BY ANY SUCH CONSUMER PROTECTION LAWS AND REGULATIONS WHERE THE PRODUCT WAS PURCHASED ARE PRESERVED. ALL MACLAREN WARRANTIES DO NOT EXCLUDE, LIMIT OR SUSPEND ANY OF YOUR RIGHTS AS A CONSUMER.

THE MACLAREN LIMITED WARRANTY WILL COMPLY WITH, AND MAY EXCEED, THE MINIMUM LEGAL REQUIREMENTS OF ANY CONSUMER PROTECTION LAWS OR REGULATIONS IN THE COUNTRY OR STATE OF PURCHASE; THESE MAY VARY FROM COUNTRY TO COUNTRY AND STATE TO STATE. ALL MACLAREN WARRANTIES ARE STRICTLY GOVERNED BY AND CONSTRUED UNDER THE LAWS OF THE COUNTRY OR STATE IN WHICH THE PRODUCT PURCHASE TOOK PLACE.

YOU CAN IDENTIFY THE SPECIFIC WARRANTOR UNDER ALL MACLAREN WARRANTIES ON OUR WEBSITE **maclarenbaby.com** ACCORDING TO THE COUNTRY OR STATE IN WHICH THE PRODUCT WAS PURCHASED. IF IN ANY DOUBT CONTACT OUR CUSTOMER SERVICES AT **customercare@maclarenbaby.com**.

MACLAREN'S LIMITED EXISTING WARRANTY MAY SUPERCEDE ANY LESSER WARRANTIES THAT ARE ASSOCIATED WITH YOUR TRANSPORT CHAIR AT THE TIME OF PURCHASE. MACLAREN RESERVES THE EXPRESS RIGHT TO VARY ALL WARRANTY TERMS AND CONDITIONS AT ANY TIME. ANY VARIATIONS ARE MADE AT MACLAREN'S SOLE DISCRETION. YOU ARE ADVISED TO CHECK THE LINK BELOW SHOULD YOU HAVE A WARRANTY QUERY TO ENSURE THAT YOU ARE SUPPLIED WITH THE MOST UP TO DATE WARRANTY TERMS AND CONDITIONS.

PLEASE CHECK OUR WEBSITE **maclarenbaby.com** FOR THE LATEST INFORMATION.

GENERAL INFORMATION

MAINTENANCE AND REPAIR

- Regularly check all rivets and connecting devices for tightness and security. Inspect all brakes, wheels and tyres and replace or repair if necessary. Check all safety devices for correct operation, in particular the primary and secondary locks. These should be free to move at all times. Never continue to use a product that is not structurally sound.
- We recommend servicing every 12 months.
- If wheels squeak, lubricate sparingly with silicone coating.
- Do NOT use oil or grease based products, as this will attract dirt, which will clog the movement.
- Only Maclaren replacement parts should be used. It may be unsafe to use parts not supplied by Maclaren.
- Accessories which are not approved by the manufacturer shall not be used.

CLEANING

- Clean frame with a damp cloth and a mild detergent, do not use abrasives and dry thoroughly.
- If the chassis parts have been exposed to salt water we recommend that they be rinsed down with fresh (tap) water as soon as possible afterwards.
- The hood may be sponged lightly using a damp cloth and a mild detergent.
- The seat cover may be washed – see care labels for details and dry thoroughly before reuse.
- Do not fold or store the product while wet and never store in a damp environment as this can cause mildew to form.

It is important that great care is taken to ensure that children are kept well clear of the transport chair when folding or unfolding the product. Finger traps and scissoring actions are unavoidable during these operations.

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
maclarenbaby.com

IMPORTANT – WARRANTY WILL BE VOID IF INFORMATION
DEFACED OR REMOVED FROM BUGGY, CARTON OR THIS
INSTRUCTION LEAFLET

PLEASE HELP US SUPPORT THE ENVIRONMENT BY RECYCLING

Maclaren encourages consumers to recycle their old Maclaren Buggys. Maclaren Buggys can be dropped off, or shipped at owner's expense, to any of Maclaren's four recycling locations below.

CUSTOMER SERVICE:

☎ Maclaren
Station Works, Long Buckby
Northamptonshire NN6 7PF
United Kingdom

🌐 maclarenbaby.com

The words **MACLAREN**, **MAJOR**, **MAJOR ELITE** and associated logos are trade marks.
© Maclaren. All Rights Reserved.

MS12302 MJ.1113MJGLO